
earis[®]
MAX

DE: Mikrofon / Sender
EN: Microphone / Transmitter
FR: Microphone / Émetteur
NL: Microfoon / Zender
ES: Micrófono / Transmisor

Additional languages are available on
the Humantechnik website:

<https://www.humantechnik.com/produkte/tv-hoersysteme/earis-max-das-system/earis-max-bluetooth-mikrofon/>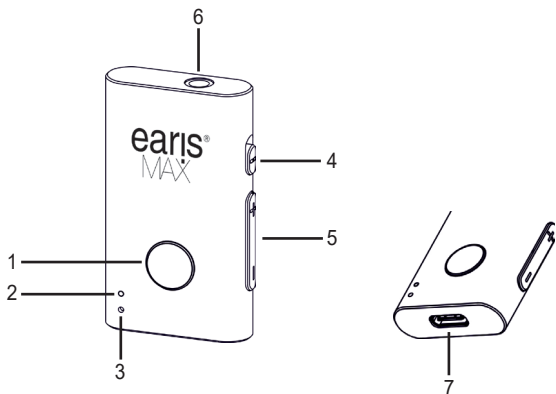

1	Funktionstaste / Funktion button / Bouton de fonction / Functietoets / La tecla de función
2	Status LED / Status LED / LED d'état / Status-LED / El LED de estado
3	Mikrofon / Microphone / Microphone / Microfoon / Micrófono
4	Umschalter: Sampling Rate 24kHz/48kHz / Sampling rate 24kHz/48kHz / Taux d'échantillonnage 24kHz/48kHz / Bemonsteringsfrequentie 24kHz/48kHz / Frecuencia de muestreo 24kHz/48kHz
5	Lautstärke / Volume / Volume / Volume / Volumen
6	3.5mm Audio Eingang / 3.5mm Audio Input / Entrée audio 3.5mm / 3,5mm audio-ingang / Entrada de audio de 3,5 mm
7	USB-C Ladeanschluss, USB-Audio-Eingang / USB-C charging port, USB audio input / Port de charge USB-C, Entrée audio USB / USB-C oplaadpoort, USB-audio-ingang / Puerto de carga USB-C, entrada de audio USB-A

Bluetooth® Classic	
Ein-/Ausschalten. Power on/off. Allumer/Éteindre. Aan/uit. Encender/Apagar.	Funktionstaste für 3 Sekunden drücken. Press function button for 3 seconds. Appuyez sur le bouton de fonction pendant 3 secondes. Druk op de functietoets gedurende 3 seconden. Presiona el botón de función durante 3 segundos.
Bluetooth® Classic. Bluetooth® Classic. Bluetooth® Classique.	Funktionstaste 2x kurz drücken. Press function button twice quickly. Appuyez deux fois rapidement sur le bouton de fonction. Druk twee keer snel op de functietoets. Funktionstaste 2 veces cortas.
Bluetooth® verbinden. Connect Bluetooth®. Connecter Bluetooth®	Funktionstaste 2x kurz drücken. Press function button twice quickly. Appuyez deux fois rapidement sur le bouton de fonction. Druk twee keer snel op de functietoets. Funktionstaste 2 veces cortas.
Bluetooth® verbinden. Bluetooth® conectar.	

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Important note:

The Auracast™ specifications allow for different sampling rates. Most playback devices support various sampling rates. The best sound quality is achieved with a sampling rate of 48 kHz. However, it may happen that some playback devices, such as hearing aids, only support a sampling rate of 24 kHz.

If you want to use our EarisMAX microphone with such devices, you can switch the sampling rate using the toggle switch (4). This function applies only in Auracast mode.

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Technical data

Bluetooth:	5.2
Codec:	LC3, SBC, AAC
Audio Latency:	<50ms
Transmission Range:	up to 30m
Power Input:	USB-C 5VDC 100mA
RF Transmission Power:	max. 10dBm
Battery Capacity:	150 mAh
Operating Temperature:	0°C to 40°C
Weight:	10g
Size:	47x27x8mm

These devices comply to the requirements of the following EC directives:

- 2011/65/EU + 2015/863/EU RoHS-directive
- 2012/19/EU WEEE-directive
- 2014/53/EU RED-directive

Conformity with the above-mentioned directives is confirmed by the CE mark on the device. CE declarations of conformity are available on the Internet at www.humantechnik.com/service.

Humantechnik's UK authorised representative:

Sarabec Ltd, 15 High Force Road, MIDDLESBROUGH TS2 1RH
United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments.
UK declaration of conformity available from: Sarabec Ltd.

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For other service-partners
in Europe please contact:

Humantechnik Germany
Tel.: +49 (0) 76 21/ 9 56 89-0
Fax: +49 (0) 76 21/ 9 56 89-70
Internet: www.humantechnik.com
e-mail: info@humantechnik.com